

Length Measurement

How to use this sizing chart

1.

- Fold sizing chart on dotted line
- Make a 90-degree angle
- Place on hard surface
- Put fold firmly against wall
- Tape to prevent mis-sizing

2.

- Place heel of AFO on sizing chart
- Be certain to include posterior stop
- If foot is longer than brace, include foot

3.

- Trace AFOs for accuracy
- Match longest point to number on right
- If you are between or slightly above line, round up

4.

Necessary Adjustments for Circumference

For all AFOs, TCFlex, DAFOs, KAFOs, and SMOs, with:

- Solid ankle, instep trapping
Increase 1 full size
- Articulating hinges, posterior stop and instep strapping:
Increase 1.5
- Turbo boot, posterior stop, hinges, footplates, instep, forefoot and toe strapping:
Increase 2 sizes



children to adult sizes



KEEPINGPACE
Children's Orthopedic Footwear

ACOR Orthopaedic, Inc.
P (800) 237-2267
F (800) 830-8445
USA
www.acor.com
orderentry@acor.com

